

AQUAGOLD®
MICROCHANNEL TECHNOLOGY®

fine touch™



Achieve Clinical Luxury With Revolutionary Microchannel Technology

AS FEATURED IN

ELLE MAGAZINE® ((SiriusXM)) PSC NBC CBS BAZAAR MedEsthetics NEWBEAUTY
PSP REFINERY29 OBSERVER KKW FASHIONISTA marie claire GLAMOUR BYRDIE

1-800-55-fine touch
General: 702-874-8544

WWW.AQUAGOLDFINETOUCH.COM



Use of Microchannel Technology and Injectable Hyaluronic Acid for Acne Scars


Study result on treatment of acne scars using AQUAGOLD® *fine touch™* and HA presented by Dr. Erin Gilbert (Board Certified Dermatologist and Assistant Professor of Dermatology at SUNY Downstate) at the South Beach Symposium on February 12, 2016

Synopsis

Hyaluronic Acid (HA) has been commonly used for injection into the mid-to-deep dermis for correction of moderate-to-severe facial wrinkles and folds such as nasolabial folds. Using techniques largely popular in Europe and Asia, one can inject the HA into the superficial layers of the skin to improve texture and fine lines and provide an increased level of hydration and radiance.

Objective


To use microchannel technology and injectable hyaluronic acid for acne scars and overall skin health and quality.

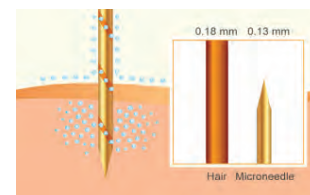

AQT-001 Single Microchannel

Results

Patient reported no discomfort, was very satisfied with the procedure and reported no downtime and immediate improvement the next day. Comparison photography was taken 48 hours after treatment and there was a significant visual improvement in the appearance of her acne scars as well as in her skin quality.

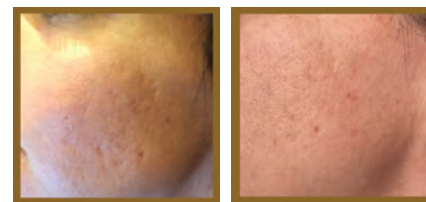

Before

After 48 hours

Methods


In this study, we used a new microchannel technology to deliver HA diluted with saline (1:4) into the superficial layer of the skin (zero to 600 µm in depth) in a 29 year old woman with moderate to severe acne scarring on her cheeks. The microchannel device is equipped with 20 micro channels that are smaller than human hair. Each microchannels made of surgical grade stainless steel plated with 24-karat gold and are 130 µm in diameter. The array of microchannels are attached to a vial that contains the HA solution. The plunger in the middle of the device releases a consistent dose of the solution. For comparison, we treated only one side of the face with a monophasic double cross-linked HA diluted in saline (0.2cc HA and 0.8cc saline) to prevent tinsel effect or lumpiness at the skin surface. Using serial, overlapping perpendicular depressions, we performed approximately 60 injections in 2 minutes which is equivalent to 1,200 micro injections. No topical anesthesia was applied.

Conclusion

Superficial microinjections of diluted HA using a new microchannel device appears to provide significant visual improvement in moderate to severe acne scarring and skin quality with no adverse effects.



AQT-001 Top View

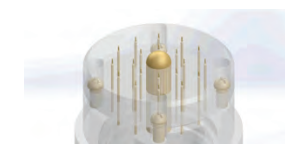

AQT-001 Technical Design

"AQUAGOLD provides the skin revitalizing benefits of microneedling less invasively. I customize a proprietary cocktail of super ingredients like hyaluronic acid and Botox and stamp delicate areas like your eyes, décolletage, and hands with it. You'll glow instantly."



-- Christopher Pavlou, MD --

"Anything we put in there can be any secret combination of things...The goal is this goes with every one of you treatments. If you're getting fillers inside, you should get fillers on the outside. So we call this the Finishing Touch. This is infusing real medicines."



-- Jason Emer, MD --

"Tried out the AQUAGOLD 24 KARAT GOLD microchannel microneedling device. I put some Juvederm and Botox in the cocktail and treated face and neck...Smooth smooth! Loving the results!! It's been a few days since I've done the AQUAGOLD treatment and I have to say I really, really like my skin. I feel like it's a little bit smoother, looks a little bit more airbrushed."



-- Anna Guanche, MD --

"Patients see immediate results – plumpness, pore size, tightness and overall skin quality! This treatment can also be performed with Botox, to help treat facial flushing and large pores, and with Filler, to help smoker's lines and under eye crepiness."



-- Jaimie DeRosa, MD --

"AQUAGOLD® *fine touch™* is a unique broad-spectrum applicator that effectively and comfortably deliver small micro droplets of treatment solutions directly into the skin enabling greater absorption. Some common combination of solution infusions are Botox, Filler, PRP."



-- William Harper, MD --

"This device delivers solutions composed of hyaluronic acid fillers, Botox, Dysport, and vitamins into the skins surface. This is a great treatment for areas such as the face, neck, back of hands and décolleté."



-- Sumit Bapna, MD --

"... at the bottom of the vial, we have a small amount of the combination of botox and hyaluronic acid filler. And the deliver cap of this has 20 microneedles that are smaller than human hair. The outside of the needles has spiral chnnnels that deliver the product in the vial directly into the skin and it's all virtually painless."



-- David Van Dam, MD --

"AQUAGOLD® *fine touch™* uses a microchannel technology to effectively deliver ingredients right into the surface of the skin. Using twenty individual microchannels that are each thinner than a human hair we are penetrating into the skin to deliver specific treatments. Can be done on Face, neck and decolette to treat any textural irregularities including: fine lines & wrinkles, large pores, acne scars or unevenness."



-- Dennis Gross, MD --

Copyright © 2014-2018 Aquavit Pharmaceuticals, Inc.
Science Personalized. Healthcare Modernized.

All registered trademark are the property of their respective owners.
AQTMARCOL8923563



